

Indication Guide for Metal Free Restorations

Products	Strength	Prep Design	Recommended Reduction	Minimum Reduction	Anterior Bridge	Posterior Bridge
Tetra-Z Zirconia (3Y)	1,200 MPa	Chamfer Margins Recommended	1.5 mm	0.6 mm Reduction	✓	✓ Max 2 Consecutive Pontics
Tetra-Z Lingual (3Y)	1,200 MPa on Biting Surfaces	Chamfer Margins Recommended	1.5 mm	0.6 mm Reduction	✓	✓ Max 2 Consecutive Pontics
Element-Z Zirconia (4Y)	1,050 MPa	Chamfer Margins Recommended	1.5 mm	0.6 mm Reduction	✓	✓ Max 2 Consecutive Pontics
Ultra-Z Esthetic Zirconia (5Y)	750 MPa	Chamfer Margins Recommended	1.5 mm	0.8 mm Reduction	✓ 1 Pontic	✓ No Molars / 1 Pontic
LiSi Press	508 MPa	Chamfer Margins Recommended	1.5 mm	0.8 mm Reduction	✓ 1 Pontic	✓ No Molars / 1 Pontic
Porcelain Fused to Zirconia	Forte 1,400 MPa	Chamfer Margins Recommended	1.5 mm	0.8 mm Reduction	✓	✓ 1 Pontic