

Surgical Guide Process

Receive full DICOM Data Set via 360 Courier from the doctor



Receive Model and / or Impression for you to scan in house or STL file from an Intraoral Scanner



Download the DICOMS from Courier to your local PC. * Create a folder on your local pc to hold these files




Upload the DICOM files from your local into 360dps via the “New Patient” button to plan the case.




Set up an online review with the doctor (ex. GoToMeeting) to review the case and get the case approved for production.

If 360imaging is printing the surgical guide for you:

Save the approved .dps file and choose “Order Guide” to fill out the patient RX form and submit the case to 360imaging.




360imaging will process the case (sleeve and print), and then ship the guide with all the paperwork back to the lab.



You will receive the guide and send it with the paperwork (Drill Protocol Sheet, Print Report and Invoice) to the doctor.

If you are printing the surgical guide in house:

Once the doctor has approved the case, you will sleeve the case in 360dps and produce all of the paperwork (Drill Protocol Sheet & Print Report) and STL print files for the guide & model.



Finally you will print the guide and model and ship it to the doctor along with the paperwork.